




EYETABLET

... a unique and simple CHART PANEL to meet all professional needs in vision testing !

High Technology, User-Friendly Touchscreen Interface

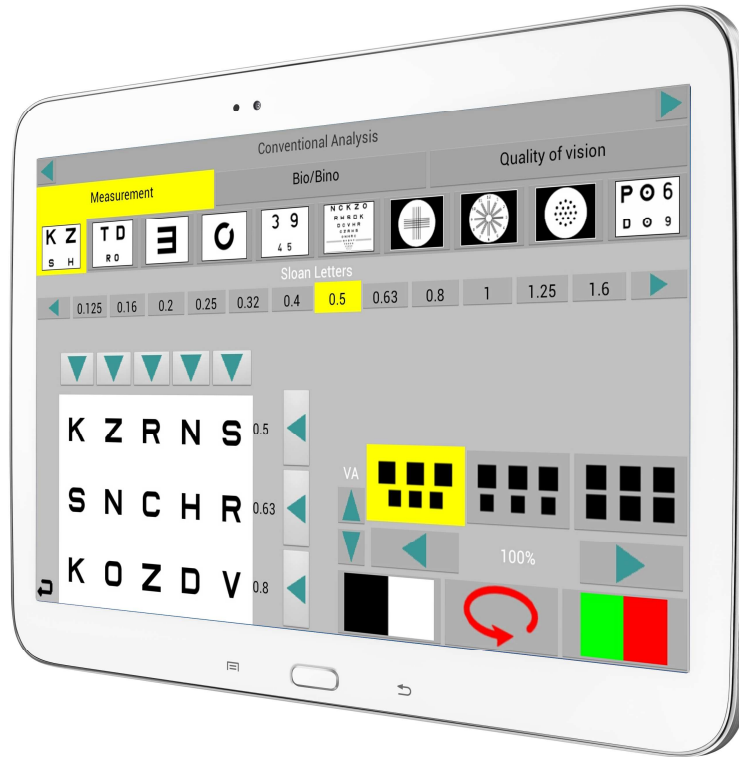

TWO APPLICATIONS IN ONE DEVICE

OPERATION PANEL

Another remote control to have a possibility to manage your VistaVision

NEAR VISION TESTING

to check the near vision with three test programs (Reading, Measurement, Quality of Vision)

Change the way you work: the OPERATION PANEL provides a perfect control of your Far Vision Device!

- The professional way to interact to your VistaVision LCD chart system
- Direct control on all parameters of a test
- You can show a test to the patient without look directly to the chart panel
- This networking technology allows you to use multiple chart panels in the same examination room
- Simple and intuitive
- Fast built-on and instant-off (Android 4.4 Operating System)
- Complete, precise and versatile software; programmable patient's vision examination
- Free software update with micro SD Card (optional, but recommended) forever
- Long-life, up to date technology and strong competitive price !

A new way you can work: NEAR VISION TESTING with accurate and easier tests!

- Application to check the near vision (40cm)
- Examinations programs:
 - Measurement: Duane, Adult/Child grids, Red and green, Snellen, Numbers, Landolt's ring, Tumbling E, Kolt, Circular optotypes, Cross cylinder dots, Half clock dial, Astigmatism clock dial, Binocular balance, Circular Biocular optotypes, Jaeger, Column/Row bright letters.
 - Reading: NVREAD, Newspaper, Phone directory, Broadcast programming, Patient information leaflet, Bus timetable, Map, Sheet music
 - Quality of Vision: Point of fix, Worth four dot test, Anaglyph, Cross test, Pointer test (vertical/horizontal and double), Rectangle test vertical/horizontal, Stereo Triangle (St), Stereo Balance (V), Binocular vision, Cross test, Mallet test, Osterberg Stereo test, Color blindness test, Amsler's grids.
- Just a touch on the screen to go directly to a specific test

Controllable test list


Snellen Letters, Sloan Letters, Numbers, Children's Images, Tumbling E, Landolt's Ring, HOTV, ETDRS tables (1, 2, R), LeaLike Symbols, Allen Pre-school, Circular Optotypes, Kolt Symbols, Contrast Sensitivity Chart (6 lines, 12 lines), Hebrew Letters, Cross Test (K), Pointer Test (Zv, Zo, DZ), Rectangle Test Vertical (H), Stereo Triangle (St), Stereo Balance (V), Differentiated Stereo (D6/D8), Cowen (C), Random Dot Test (RS, RH), Rectangle Test Horizontal, Binocular Balance (1, 2, 3), Osterberg Test, Schober's Test, Phoria Horizontal-Vertical, Mallet Test (OXO), Polarized Clock Dial, Osterberg Stereo Test, Cross Test, Contrast Sensitivity Test, Sinusoidal Grating, Color Blindness Test (Ishihara), Red and Green, Worth Four Dot Test, Astigmatism Clock Dial, Cross Cylinder Dots, Amsler's Grid, Random Dot Test Red-Green, Adult Grid, Child Grid, Row Bright Letters, Column Bright Letters, Half Clock Dial, Point of Fix, Maddox LED Test, Pictures for Retinoscopy, Photostress Recovery Time, Twilight Vision Test.

Some example to control the chart panel

- Touch screen interface offers great convenience for operation
- Just a touch on the screen to go directly to a specific test
- Graphical representation on screen guides test process in easy and fast mode
- Simple display sequences and improved tests navigation
- Simple masking buttons offers selection of a single letter, an horizontal or descending line
- Buttons and slider allow to control visual acuity and contrast
- Direct access to Polarisation Inversion, Red/Green and Black/White when available for the test
- One button to change the crowding on letters, numbers, symbols and pictures tests

Hardware Specifications

- LCD color **touch-screen** 10.1-inch, 1280x800 pixel
- Stylish Design with suitable stand
- Dimension (WxHxD): 247 x 171 x 8.9 mm
- Weight: 500 g
- Internal battery: 7200mAh Li-ion Battery
- Operation Panel Connection Mode: Wifi
- Charger unit: DC 5.0V/2.0A

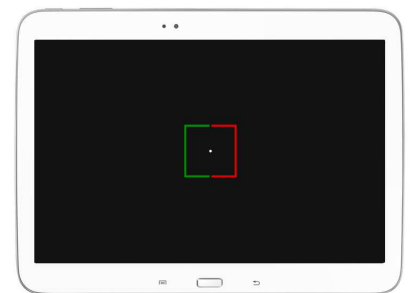

How to order the eyeTablet?

To order : "eyeTablet10v3 Wifi" use the code: **4DMD EYETABL-3W**

VISTAVISION® is a registered trademark of DMD Computers s.r.l.



VistaVision® and eyeTablet are designed, engineered and produced by DMD Computers srl

Via Monviso, 14 – 10090 Villarbasse Torino Italy; Tel ++39 0119528282; Fax ++39 0119528402

www.dmdmedtech.com , medtech@dmd.it

DMD
COMPUTERS